Anleitung

Art.Nr. 2096520

Pedalo® - Vestimed 50

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14004050

Pedalo® - Vestimed 50

Before using the device for the first time (or before starting with assembly), please read instruction completely and keep it

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Tel. +49 (0) 7381 - 9357 0
Fax +49 (0) 7381 - 9357 40

info@pedalo.de
www.pedalo.de

Important information and warning hints:

Warning! This device is a very demanding one! Especially when used for the first times and/or when doing exercises in standing position, a special degree in caution, attention, circumspection and a certain degree in motor abilities is assumed. In case of doubt first use the device only with assistance and/or appropriate protective equipment.

Warning! The packing is not part of the product and have to be disposed.

Warning! The pedalo® Vestimed® is only to be used for the purposes described in this user manual.

Warning! Place the unit only on flat level ground.

After longer or frequent use, check that the screws and all installed parts are tight and retighten if necessary. Defective and worn parts should be replaced by original replacement parts immediately. Protect item for damp and wet.

The power belts may be stretched up to 100 %. Will they further stretched, the bands can be damaged or broken. This risk of serious injury by quickly back strap end.

Protect the power belts from heat, dust and direct sunlight
Avoid contact with sharp objects, nails

This device is meant for using from persons at the age of 3 up.

Maximum weight capacity approx. 120 kg

Powerbands can be easily attached through the holes of the board. Insert the loops from below through the bores in the upper board. Now use the carabines to hang the Powerband in the loop. The textile Powerbands are featured with 9 loops for hands and feet for individual use in optimal length and resistance. Basic length 140 cm, washable, 0.2 kg

Optional accessory (not included in delivery):

Pedalo® Training Stick (w/o illustration). Item no. 144 140

The Pedalo® Training Stick is used for a variety of exercises. Strengthening in conjunction with resistance bands, stretching, balance and gymnastics.

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Holz-Hoerz GmbH